User's Guide





Model Shown : 3050-60



Electric Examination Couch 3050-60

Product Identification

Model Number :	Serial Number :
Date of Purchase :	Name of Owner / Facility :
Name of Dealer :	Dealer's Phone Number :
Promotal Authorized Service Company :	

Legal Notice

PROMOTAL

22, rue de Saint-Denis de Gastines B.P. 26 - 53500 ERNÉE Cedex

FRANCE

Tél. : +33 (0)2 43 05 12 70 Fax : +33 (0)2 43 05 68 99 internet : www.promotal.com The descriptions and specifications contained in this Operating Manual are deemed correct at the time of printing.

Promotal, however, reserves the right to modify its models and its procedures or render them obsolete without notice.

Before any order, we recommend that our customers consult a local sales manager.



Summary

Important information	6
Safety symbols	6
Applied parts	7
Electromagnetic interference	7
Unpacking precautions	8
Medical device delivered on a wooden pallet	8
Storage conditions	8
Conditions of use Unpacking and Installation	8 8
Step by step	0 8
Check	9
Remark:	9
Authorised EC Representative	9
Cleaning protocol	10
Warning	10
Cleaning/Disinfecting	10
User manual – iQuest	11
Intended purpose	11
Patient weight capacity	11
Protection against penetration of liquids	11
Characteristics	11
Electrical connection	12
Dimensions / Installation precautions	12
Assembling instruction	13
Assembling the upholstery	13
Upholstery Backrest	13
Assembling the support handle	14
Upholstery Seat	14 15
Assembling the side casings	
Safety	16
Perimeter protection for the user	16
Using the individual control	18
Manual command	18
Presentation	18
Adjusting the height	19
Adjusting the trendelenburg position Adjusting the backrest tilt	19 20
Position QE1 et QE2	20
Welcome position	20 21
«Emergency» position	21
	22
Using the Couch	
Paper roll Installation of the teston well helder	22 22
Installation of the paper roll holder Using the paper roll	22 22
Mechanical braking on individual and steering wheels	22
Using the brakes	23
Using the steering wheel	23

Summary

Accessories	24
Pair of Clamps	24
(reference 166)	24
Arm rest	24
(reference 2536)	24
Pair of stirrups	25
(reference 850)	25
Installing the stirrups	25
Pair of legrests	25
(reference 840)	25
Installing the legrests	25
Pair of covers for legrests	25
2 hook IV pole	26
(reference 985-01)	26
Installing the IV pole	26
Auto-blocking I.V. pole	26
(reference 2985-01)	26
Installing the IV pole	26
Adjusting the height of the IV pole	26
Headrest	27
(reference 1102-10)	27
Backrest extension	27
(reference 2055-01)	27
Installating the Backrest extension	27
Adjustable paper-roll holder	28
(reference 2056-02)	28
Installating the paper roll holder	28
Installation of the paper roll	29
Side foot rails	29
(reference 30878-65P, 30878-75P, 30878-85P)	29
Positioning the rails	29
Rails coté tête	29
(reference 30878-65T, 30878-75T, 30878-85T)	29
Positioning the rails	29
Vise	30
(reference 878-12)	30
Crystal head section side protectors	31
(reference 3056-02)	31
Positioning the protector	31
Crystal foot section side protectors	32
(reference 600150-65C for flat upholstery 600150-65	32
reference 600150-75C for flat upholstery 600150-75)	32
Positioning the protector	32
The pair of barriers	33
(reference 85114-01)	33
Installing the pair of barriers	33
Using the pair of barriers	33
Push handles	34
(reference 3095-01)	34
Setting up the handles	34 35
Foot pedal	35

Presentation	35
Using the pedal control	35
Setting up the control unit	36
Options	37
Accumulator	37
(reference 3092-01)	37
Charging the accumulator	37
Accumulator safety symbols	37
Technical characteristics of the accumulator	37
Circuit breaker	38
(reference 3058-01)	38
Using the circuit breaker	38
Reel for electric cable	38
(reference 3032-01)	38
Supplemental power socket	39
(reference 30400-01)	39
Using the supplemental power socket	39
Equipotential terminal	40
(reference 2059-01)	40
Notes	41
Lifespan of the Medical Device	44
The wear parts are:	44
Compulsory / specific maintenance	44
Once a year, ensure that the following checks are carried out by a	aualified technician (contact vour
dealer):	44
Once a month :	44
Medical device end of service life	44
Maintenance notebook	45
Warranty Information	46
Warranty	46
Obligations	46
Exclusions	46
Exclusive Remedy	46
No Authorization	46

Safety symbols



Warning sign

Information marked with this symbol must be read and strictly complied with!

Remark

Draws your attention to a procedure, practice or situation.



Humidity limitation







Atmospheric pressure limitation



Temperature limitation

objects



transportation

Manufacturer

Protective earth

Equipotential terminal

Use in a dry place only

Correct orientation for



Keep in a dry place



Not to use sharp



Type B applied part

Life cycle (motor cycle)



Dangerous voltage / risk of electrocution

1 minute working / 9 minutes stopped

manual / brochure

Refer to the instruction



To be disposed off separately from household waste



CE marking

General safety sign

Housing protection

classification

IP

Remark

the equipotential terminal allows the connection of an equipotential conductor to bring all the different elements of an electro-medical system to the same potential.




Do not sit on the back rest



Do not sit on the leg rest



You must not use the medical device for transfers

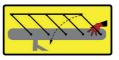



Warning

You must not remove the pictograms and warning signs provided by the manufacturer! The manufacturer disclaims all responsibility in case of removal of these signs.

Applied parts

The applied parts according to standard EN 60601-1 are:

- PVC upholstery
- Assistance handles (accessories)
- Gynecological examination stirrups (accessories)

Electrical power supply

The equipotential terminal must not be used as a protective earth connection under any circumstances.



This medical device has electrical classification 1, it must only be connected to a power supply equipped with a protective earth.

Electromagnetic interference

This Promotal medical device was designed and built to minimize electromagnetic interference with other equipment. If interference is, however, observed, you must remove the apparatus causing the interference from the room and/or plug it into an isolated circuit.

Medical device delivered on a wooden pallet

The medical device positioned on a wooden pallet may be easily moved using a forklift truck, as long as this is used correctly. Before transportation, ensure that the forklift truck is correctly positioned in relation to the pallet, and that the unit is stable.

Storage conditions

Room temperature: Relative humidity : Atmospheric pressure : -15 °C to +60 °C (+5 °F to 140 °F) 10 % to 90 % (without condensation) 0.5 bar to 1.05 bar (500 hPa to 1050 hPa)

- Do not stack material.

frequent passage.

final destination.

All storage must be carried out in accordance with the following recommendations:

- Clean, aired and temperate area.

 Medical device stored in an area sheltered from bad weather and direct sunlight.

- Dry room.

- Medical device protected from shocks.

Conditions of use

- Dry and temperate area.

– Maximum altitude : 2000 m

Unpacking and Installation

Step by step

1) During unpacking, remove all staples and remove the cardboard packaging carefully.

Caution : be careful with cutting tools, as fragile parts of the medical device *(covering, plastic housing, etc.)* may be near them.

2) If possible, transport the medical device on its pallet up to the final place of use.

The medical device is adjusted to a resistant position for transportation.

see: as indicated in the diagram

3) Cut the two green bands that fix the chair to the pallet..

4) Next, take the chair off the pallet.

Caution Four people are required to remove

the medical device from the pallet.



– Keep in its original packaging until the ination.

- Do not store in an area subject to

- Temperature 10 to 40' C

- Relative humidity 75% maxi.

Check

Having unpacked the medical device, follow these steps:

1) Check the delivery documents to ensure that the delivery is complete.

2) Check the external components for any

damage during transportation.

3) Check that the packaging contains the medical device, accessories and Optionss, the supply cable *(if electrical MD)* and the User Guide.

Remark:

Authorised EC Representative

Within the European Union, all problems, complaints or questions should be addressed to the Authorised EC Representative of Promotal indicated below:

Promotal 22, rue de Saint-Denis de Gastines 53500 Ernée, FRANCE Telephone : + 33 (0)2 430 517 76 Fax : + 33 (0)2 430 572 00

Warning

It is vital to read the user's manual thoroughly before manipulating this Medical Device. The equipment should only be used for its intended purpose as described in our documentation. Installation and connection must only be carried out by qualified personnel. The electrical components (cylinder, box, control handle, battery, adapter, etc.) must not be opened under any circumstances. PROMOTAL shall not be held liable for any damage resulting from non-compliance with these instructions.

Any modification to the medical device without written authorization from the manufacturer is forbidden.

Caution

Only accessories designed and provided by Promotal for this medical device are authorised for use.

Caution This medical device is not intended to be cleaned in a washing tunnel.

Cleaning/Disinfecting

This medical device must be regularly cleaned using the appropriate detergent products and disinfected using bactericidal, virucidal and fungicidal disinfectants.

A mild detergent such as soapy water can be used for routine cleaning of upholstery, stainless steel, aluminium or painted surfaces, plastic parts and control components, followed by effective rinsing and thorough drying. Detergents and disinfectants designed for use with medical appliances, such as those containing quaternary ammonium compounds, hydrogen peroxide, ethanol, chlorine compounds, etc. can be used on our medical devices provided that:


- The concentration prescribed by the suppliers of such products are complied with;
- The application conditions (contact time, quantity used, temperature, rinsing, etc.) are complied with;
- The supplier's instructions state that the detergent-disinfectant used is suitable for use with:
 - PVC, ABS, Polyamide, Polyurethane, Polypropylene
 - Epoxy-coated metal surfaces
 - Stainless steel or aluminium metallic surfaces.

Warning:

- Solvents are strictly prohibited.
- The use of abrasive powders or any other abrasive product should be avoided.
- High-pressure cleaning is forbidden.



Under no circumstances shall Promotal be held liable under warranty for any damage caused by non-compliance with the use instructions for a detergent-disinfectant.

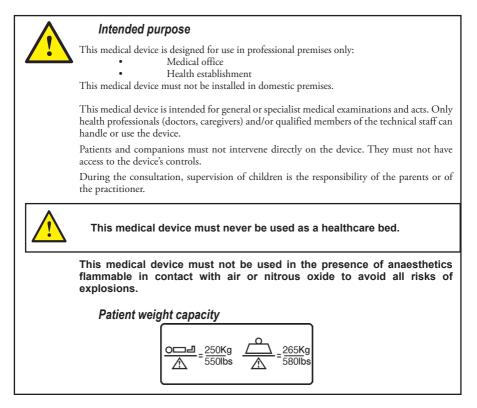

Protection against penetration of liquids

• IP X1

Characteristics

Electric variable height from 46 to 93 cm. Steel frame with matte white epoxy finish Gas-spring assisted or electric backrest Flat upholstery with class M1 flame retardant covering Hand control Adjustable levellers under base 4 ø 125 mm independent casters 1 steering castors at the head end Paper roll holder (max length 50 cm - ø 20) Mobile hoist compatibility – clearance of 12 cm



Electrical connection

This medical device must be connected to the mains supply.

- Connection to the mains supply :
- Frequency :
- Protection classification :
- Absorbed power :
- Intermittent operating mode :

120~V~or~230~V~(depending on the country) 50/60 Hz


Class 1/ type B device 575 W 1 min / 9 min



Caution


a power cut could prevent the patient support from being lowered to the low position. The patient exit must be carried out in the best possible conditions of safety.

Dimensions / Installation precautions

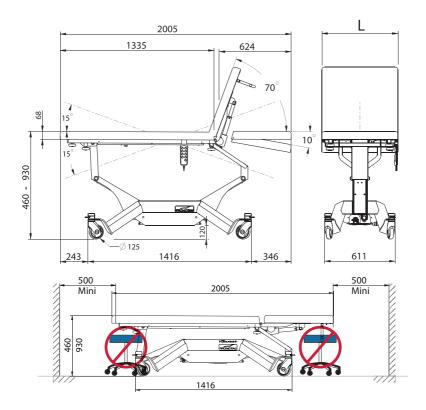

The table is delivered dismounted.

The packaging contains 2 boxes : the upholstery, the paper roll holder. Steel frame with matte white epoxy finish

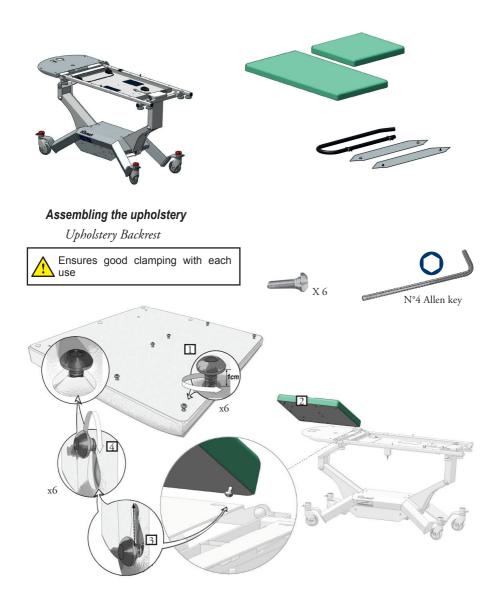

Assembling the support handle

Select the corressponding hard control support to the relevant upholstery :

- A2A200-03 / 600150-65, 600151-65

Upholstery Seat

- A2A200-04 / 600150-70, 600150-75, 600151-75

The support handle, located beteween the upholstery and the frame, may be moved or installed on the other side. Remove the mounting screw from under the frame to do so.

Ensures good clamping with each **A** X 4 use N°4 Allen key X 2 (1) 1 x4 2



A2A200-04

A2A200-03

Assembling the side casings





Caution

To avoid malfunctions and for safety reasons, no objects must be left under the chair's seat or between its moving frames.

Caution




the supply cable for this medical device may represent an obstacle and cause falls. Do not forget its presence when moving around the device once connected to the mains supply.





Perimeter protection for the user

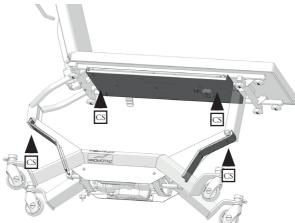

To avoid the risk of pinching between the upper frame and the base of the medical device, the couch has safety sensors (CS) that stop all movement in the event of pressure.



Safety note

Before using the medical device lowering control, ensure that no objects or obstacles are between the moving parts and the ground.



Safety note

When using the position memory function, movements can be interrupted by pushing on one of the command buttons for the couch or, as a last resort, by disconnecting the power supply cord.



Safety note

When using the pre-programmed positions, default QE1, QE2, chair position, couch position), do not leave the patient on the couch without supervision.



Safety note

Unplug the power supply cable before moving the MD.


Manual command

Presentation



Safety note

When using the pre-programmed positions, (welcome, QE1, QE2, chair position, couch position), do not leave the patient on the couch without supervision.

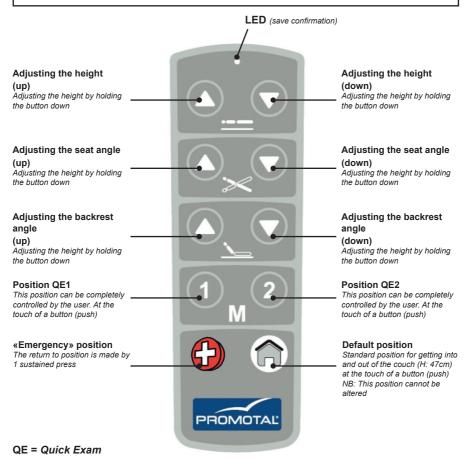



Safety note

When using the position memory function, movements can be interrupted by pushing on one of the command buttons for the couch or, as a last resort, by disconnecting the power supply cord.

Adjusting the height

The position below is obtained by pressing on the manual control.



Adjusting the trendelenburg position

The position below is obtained by pressing on the manual control.



Adjusting the backrest tilt

The position below is obtained by pressing on the manual control.



Position QE1 et QE2

The QE1 and QE2 buttons are factory-set but cannot be modified. To enter a new position for buttons QE1 and QE2, simply adjust the couch to the desired position and then press and hold the QE1 (or QE2) button for 6 seconds.

This is stored once the LED located above the command bar blinks.

The position can now be recalled by pressing once on the QE1 or QE2 button.

NB: This position cannot be modified. It is strictly prohibited to use the medical device to transfer patients.

It is strictly prohibited to load the medical device when it is on castors.



Welcome position 🍙

The position can be recalled by pressing once on the button **(***R***)**. *NB: This position cannot be modified.*



«Emergency» position ()

The return to position is made by 1 sustained press on the \bigcirc .

Return to a standard position by pressing once on the button $\widehat{\mathbf{n}}$.

NB: This position cannot be modified.


Warning It is stric patient al

It is strictly forbidden to leave the patient alone and unattended in this position.

We decline any responsibility in case of non respect of this clause




Using the Couch

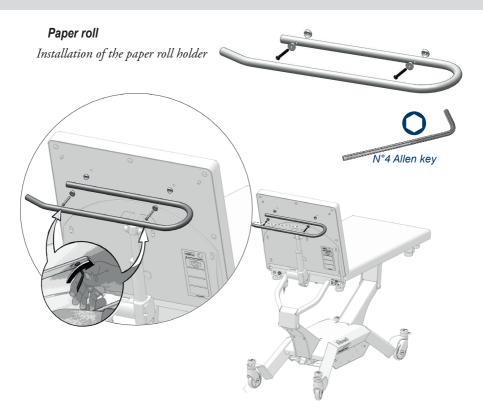

Using the paper roll Cover the upholstery with paper before use.



Caution follow the indications provided well obtain a greater longevity of the material.

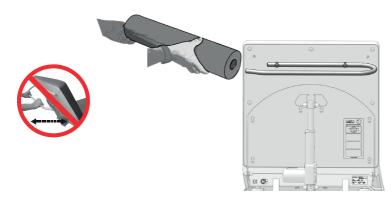

Mechanical braking on individual and steering wheels

Using the brakes

Braking position: Press on the three red buttons and the grey button Transport position: Press on the four green buttons

Using the steering wheel Steering position: Press on the grey button and on the green buttons on the other three wheels




Caution Ensure brakes are fully engaged before installing a patient.



Only accessories designed and provided by Promotal for this medical device are authorised for use.

Pair of Clamps

(reference 166)

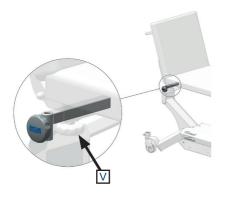

Installing the Clamps

Take the stopper (B) out of the square tube if necessary. Loosen the nuts (V), and put the pair of clamps ref. 166 (E) into position in the square tubes. Tighten the nuts.

The clamps accept accessories with a diameter of 16 mm.




Arm rest

(reference 2536)

Adjusting the arm rest

The standard arm rests can be adjusted in height (H) and can be positioned in all directions due to the ball and socket joint underneath the armrest. To adjust the armrests, loosen the screws (VS) and position as desired.

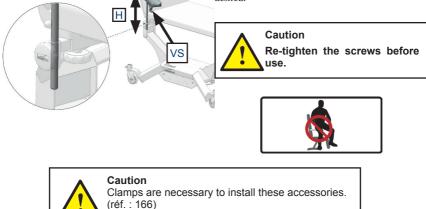


Pair of stirrups (reference 850) Installing the stirrups Ensures good clamping with each Caution



use

(reference 840) Installing the legrests

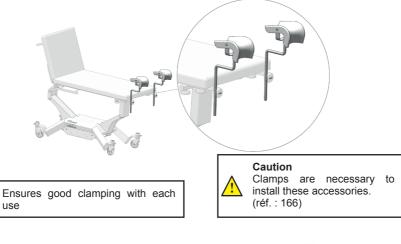

Pair of covers for legrests (reference 861)



Clamps are necessary

install these accessories.

(réf. : 166)

to




Only accessories designed and provided by Promotal for this medical device are authorised for use.

2 hook IV pole Caution (reference 985-01) Clamps are necessary to install these accessories. Installing the IV pole (réf.: 166) Ensures good clamping with each use

Auto-blocking I.V. pole (reference 2985-01)

Installing the IV pole

Unscrew the screw handle. Insert the I.V. pole and rescrew.

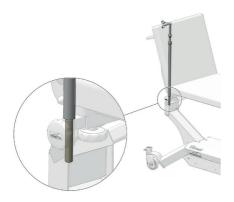

Adjusting the height of the IV pole


The IV pole has 1 sliding stem. Use the screw to adjust the height of the lower stem and the bolt to adjust the upper stem.



Caution Clamps are necessary to install these accessories. (réf.: 166)

Ensures good clamping with each use



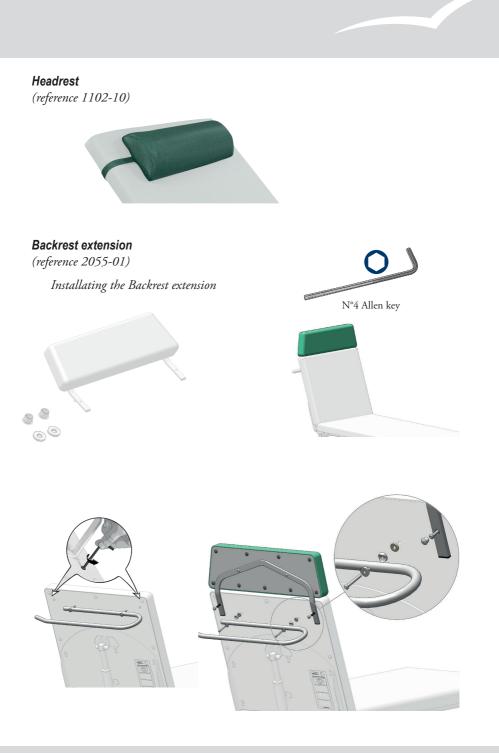



Only accessories designed and provided by Promotal for this medical device are authorised for use.

Adjustable paper-roll holder

(reference 2056-02)


Installating the paper roll holder



1. Remove the standard paper roll holder.



2. Replace the paper roll holder.

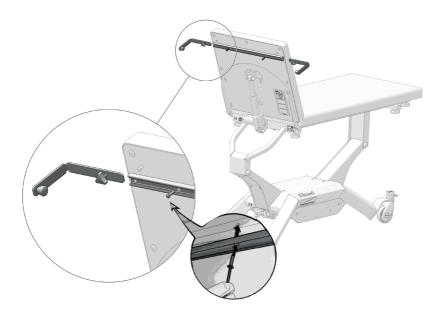

Installation of the paper roll

The paper roll is positioned on a mobile axis with adjustable hooks at each end.

- 1. Unscrew the screw handle (VS).
- 2. To remove the paper roll holder 1/2 slightly (PR).

(reference 30878-65P, 30878-75P,

Positioning the rails

3. Insert the paper roll.

Side foot rails


30878-85P)

Remove the seat upholstery.

Replace the seat upholstery.

Install the rails.


Cover the upholstery with paper before use.

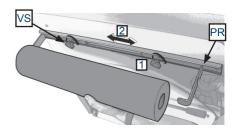


Rails coté tête

(reference 30878-65T, 30878-75T, 30878-85T)

Positioning the rails Remove the seat upholstery. Install the rails. Replace the seat upholstery.

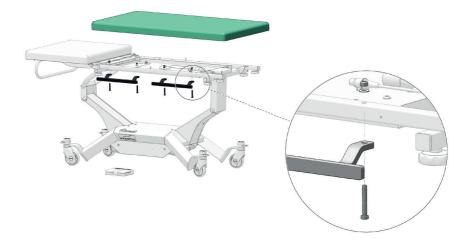

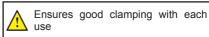

User's Guide







Only accessories designed and provided by Promotal for this medical device are authorised for use.

Vise

(reference 878-12)

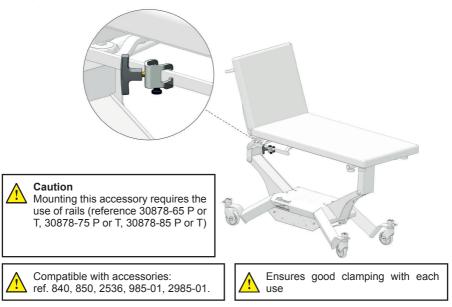

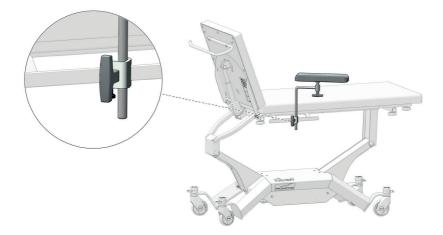

Crystal head section side protectors

(reference 3056-02)



Cruciform screw driver


Positioning the protector

1. Fixer les boutons pressions à l'emplacement prévu à l'arrière de la têtière.

2. Clipser les boutons pressions.


3. Positionner la protection sur la sellerie au niveau de la tête.




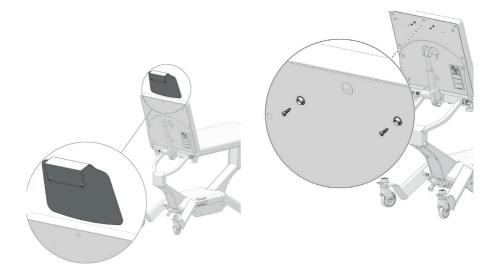



Only accessories designed and provided by Promotal for this medical device are authorised for use.

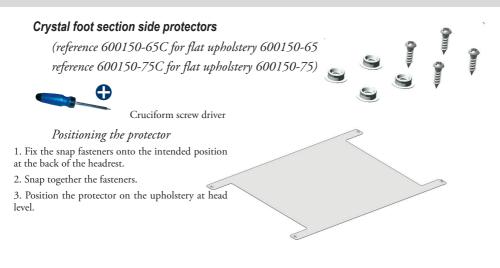

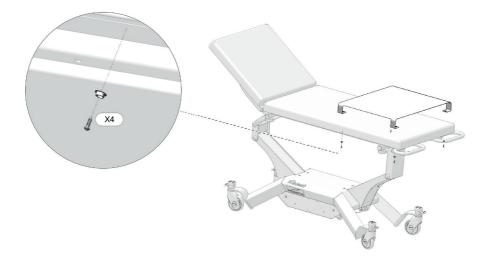

The pair of barriers


(reference 85114-01)

Installing the pair of barriers

1. Unscrew the screw.

3. Tighten the screw.

2. Insert the pair of barriers. *Approach the barriers to obtain a spacing of 25 mm beteween the barriers and the upholstery.*

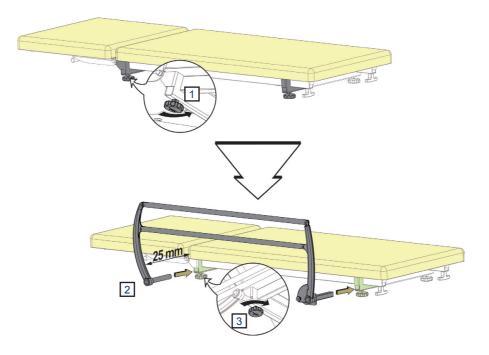


Using the pair of barriers To raise the pair of barriers

Take hold of the upper part, and lift it to its maximum height until it clicks into place.

To lower the pair of barriers


Hold the rail with one hand, and activate the lever (CB) to unlock the system with the other hand, then guide it down to its lowest position.

Ensure the side rails are correctly locked in position before use.

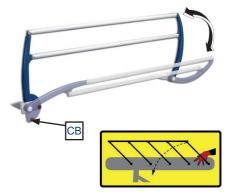



Only accessories designed and provided by Promotal for this medical device are authorised for use.

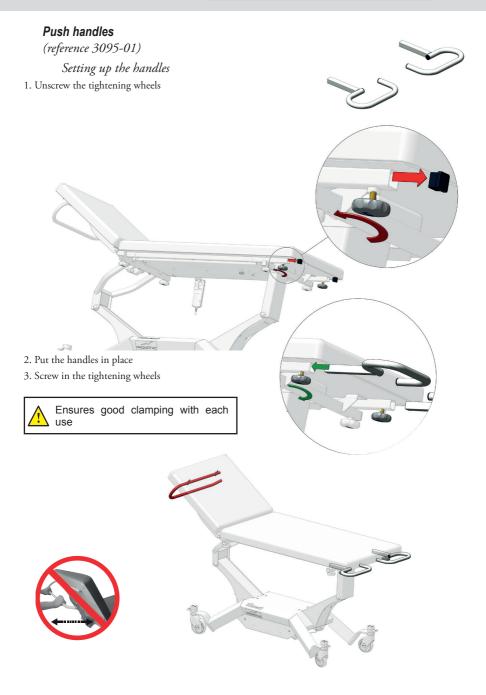

Foot pedal

Presentation



Safety note

When using the pre-programmed positions, (welcome and QE1), do not leave the patient on the couch without supervision.



Safety note

When using the position memory function, movements can be interrupted by pushing on one of the command buttons for the couch or, as a last resort, by disconnecting the power supply cord.

Using the pedal control

Simply pressing and holding on the control push button activates the desired movement.

The movement stops once the button is released. The couch is equipped with a built-in device to limit the upward and downward positions.





It is possible to select pre-programmed positions using the pedal control. Note : These positions are factory-set and cannot be modified by the user.


Press twice on the left side of the lift pedal to select examination position QE1.

Press twice on the left side of the lift pedal to select the welcome position \square .



Only accessories designed and provided by Promotal for this medical device are authorised for use.

Setting up the control unit

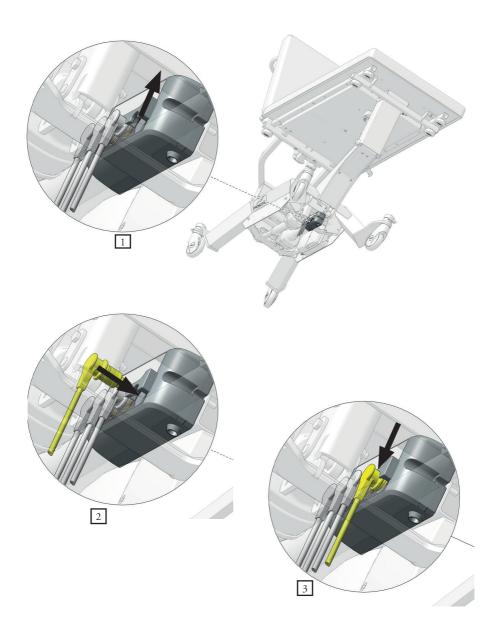

Accumulator

(reference 3092-01) (Built in to the device's power supply system)

(Note: the couch can functions the same in **«Battery mode**» or in **«Mains mode**».


Charging the accumulator

Before using the chair for the 1st time, it is recommended to leave it plugged into the mains for 24 hours to enable the accumulator to charge fully. *Note: The chair will lose power after (approximately) 5 to 10 operating cycles in "accu.mode".*



Accumulator safety symbols

Fuses

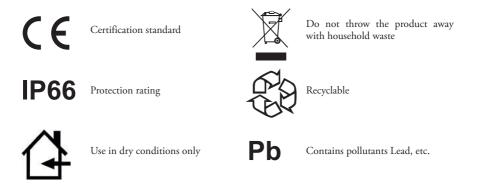

Technical characteristics of the accumulator

Voltage:	25,6 V DC
Amperage:	4 Ah
Accumulator type:	Li-Ion
Voltage (maximum):	30 V DC
Charging time:	Approx. 8 h (depending on the power supply unit)
Useful life:	approx. 1000 cycles (depends on how the chair is used)
Discharge time:	approx. 1 an (storage)

Circuit breaker

(reference 3058-01)

Using the circuit breaker

Withdraw the key to completely immobilise the couch.

Re-insert the key to use the couch again.



Supplemental power socket

(reference 30400-01)

Available exclusively and standard EU & UK



Safety note

When a medical device is connected to the supplemental power socket, the system becomes an medical electrical device in accordance with Standard EN 60601-1. It is the responsibility of the user to ensure compliance of the medical electrical system with standard EN 60601-1 (Article 16)


Using the supplemental power socket

• Properties :

120 V or 230 V depending on the country 2.6A AC max

The power consumed in a single event must not exceed 600VA.

Options 30400-01 (supplemental power socket) consists of 2 sockets, one on each side of the base of the couch.

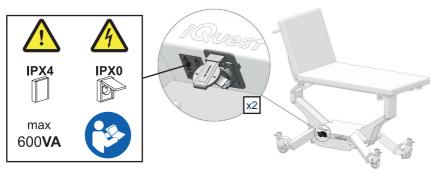

When simultaneously using products 30400-01 & 3058-01 (supplemental power socket and the circuit breaker), the sockets remain continuously powered, even if the safety key is withdrawn.


The electrical sockets are equipped with hinged covers.

When these covers are closed, the examination couch is protected against sprays of water (IP X4 according to European standard 60 529). If one of the covers is open (i.e. each time one the sockets is in use), this protection is no longer present. 1)

Equipotential terminal

(reference 2059-01)

Ce dispositif médical est de classe électrique 1. Il doit être raccordé uniquement à un réseau d'alimentation équipé d'une terre de protection.

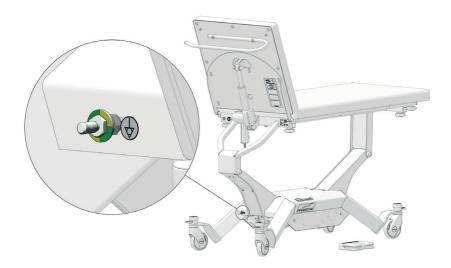

Remark

the equipotential terminal allows the connection of an equipotential conductor to bring all the different elements of an electro-medical system to the same potential.



The equipotential bonding conductor must never be used as the earth connection for protection.

Notes



This device is designed for a 10 year lifetime (except wear parts) under normal use conditions. This lifetime may vary according to frequency of use.

CHECK THE GENERAL CONDITION OF THE DEVICE AT LEAST ONCE A YEAR.

The wear parts are:

- The upholstery.
- The electric cylinders.

Promotal recommends replacing the wear parts after 5 years use maximum.

For all interventions, contact your usual dealer, indicating the Serial number of the device.



Material warning

Within the framework of maintenance, only the installation of components designed and provided for this MD by Promotal is authorised.

Compulsory / specific maintenance

Once a year, ensure that the following checks are carried out by a qualified technician (contact your dealer):

- Ensure that all screws are correctly tightened.
- Check the fixings of articulated parts.
- Ensure that the structure has not been deformed.
- Ensure the fuses are in good condition.
- Check that the power cable has not been cut or damaged.
- Check the different connections (excessive play, noise...)
- Note this information and the control date in the maintenance record.

Once a month :

• Check the correct operation of the safety devices (cf page 16).

Medical device end of service life

Your dealer is responsible for the recovery and end of life treatment of this device.

If necessary, do not hesitate to contact Promotal. We can propose solutions to treat this equipment in the best conditions.



Material warning

Upholstery whose coating is torn no longer provides an effective anti-bacterial barrier and must be replaced without delay.



Material warning DO NOT dismantle THE DEVICE

If a fault is detected, immediately contact your dealer or the dealer's technical department (cf.Warranty Chapter) for a complete diagnosis. If you have a doubt, do not use the device.

Maintenance notebook

	Type of intervention - corrective action						
Product reference: Serial number (cf. label): Manufacturing date (start date of warranty):	Name of technician						
	Date						

If the product is returned to the retailer or manufacturer, please clean and disinfect the material.

MAINTENANCE NOTEBOOK

Warranty

Promotal warrants, to the original purchaser, products manufactured by Promotal and components to be free from defects in materials and workmanship for a period of two (2) years¹ from the date of purchase.

Obligations

Promotal will, at its discretion and expense, replace defective parts reported to Promotal within the applicable warranty period, and which, upon examination by Promotal, prove to be defective.

In accordance with CE regulation, Promotal's distributor is responsible for after sales service during and after the warranty period.

Exclusions

- This warranty does not extend to :

(1) Spare parts and consumables.

(2) Travel and labour expenses.

(3) Breakdowns due to improper use, manifest neglect, or to moving the device.

(4) Equipment whose original characteristics have been changed by the user.

(5) Control units, hydraulic and electric jacks if opened by the user (seals broken).

(6) Damage, breakdowns, failures, or defects attributable to outside causes (lightning, electric surges, floods, natural catastrophes, impacts, etc.) or to the presence of foreign objects.

(7) Damage caused by improper hook-up or by the power supply, damage caused by corrosion or by gradual deterioration of the product.

(8) Indirect damage related to loss of use and penalties generated by poor performance.

(9) Aesthetic damage incurred by the outside parts of the equipment, which does not hinder proper operation, such as scratches, chips, and scrapes.

(10) Devices whose serial number has been rendered illegible or has been changed or removed

Exclusive Remedy

Promotal's only obligation under this warranty is the replacement of defective parts. Promotal shall not be liable for any direct, indirect, special, incidental, exemplary, or consequential damages or delay, including, but not limited to, damages for loss of profits or lose of use.

No Authorization

No person or fi rm is authorized to create for Promotal any other obligation or liability in connection with the products.

THIS WARRANTY IS PROMOTAL'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED.

¹ Upholstery sets are warranted 1 year from manufacturing defaults.



EU Declaration of Conformity

We,

Promotal

22 rue de Saint Denis de Gastines 53500 Ernée – France SRN: FR-MF-000001666

declare, under our sole responsibility, that the following electrical medical device:

Commercial name: iQuest 3050-60

Description: Electric Examination Couch Basic UDI-DI: 37014094IQUESTJU

is a Class I medical device,

complies with the requirements of Regulation~(EU)~2017/745 of the European Parliament and of the Council of 5 April 2017,

complies with the following European directives:

- Directive 2011/65/EU (RoHS 2) of the European Parliament and of the Council of 8 June 2011
- Delegated Directive (EU) 2015/863 of the Commission of 31 March 2015
- Directive (EU) 2017/2102 of the European Parliament and of the Council of 15 November 2017

meets the applicable European standards,

bears, as such, the CE marking.

Year that the CE (Medical Devices) marking was initially affixed: 2018

Signed in Ernée, On 26 May 2021

Rudolf MOURADIAN Chairman

 Promotal

 22 rue de Saint-Denis de Gastines, 53500 Ernée, France

 Tel.: +33 (0)2 43 05 17 76
 Fax : +33 (0)2 43 05 72 00





PROMOTAL - FRANCE www.promotal.com DIC3050-60_2121EN

